

ID: 2409

Date: 14.01.2020

Version: 5

Instruction for use regarding the products

P-100, P-300, P-400, P-500 and P-600

Protection packaging:

It is recommended that the outer packing, «bubble plastics», is kept on during transport and storage. This will protect against mechanical damages and make the product unsterile. It is assumed that the product is handled carefully by the user.

Sterilization:

The sterile containers are wrapped in a sterilization bag (peelbag). The set is sterilized with Gamma irradiation. If the sterilization bag (peelbag) is damaged the containers are considered unsterile.

Re-sterilization:

It is not recommended because of the hazard of stress cracking in the plastic material.

Freight:

It is recommended that the outer packing; «bubble plastic», is kept on during transportation.

Air freight:

In an airplane it is recommended to keep the containers in pressurized cabin. In some small airplanes the freight compartment is not pressurized, this may cause the plastic bag to expand and rupture.

Storage:

The containers should be stored in a dark and acclimatized room. Avoid direct sunlight as this may affect the plastic material.

Re-use:

The products are single use. Re-use may cause stress cracking which can lead to an unsterile product and it can be hazardous for the patient.

Lot number:

Medco's identification number.

Expire date:

We recommend that the products are used before the expiry date:

Temperature limit:

The product can safely be stored within the temperature limit from 15°C to 25°C.

Manufacturer:

Medco AS, Årvollskogen 83, 1529 Moss, Norway

When using the sterile containers we assume it is in accordance with the users work directives.

